

How To Apply a Tag

- [Cut-Band Tag Application, Removal, and Disinfection Guide](#)
- [Bracelet Tag Application, Removal, and Disinfection Guide](#)
- [Tummy Tag Application, Removal, and Disinfection Guide](#)

Cut-Band Tag Application, Removal, and Disinfection Guide



Cut-E Remo

App

1. **ASSIGN** the tag to a patient i
(See How to Store and Test a MyChild
2. **THREAD** the cut-band UP thr
of tag (white side with writin



3. **SLIP** the loop over baby's foot



Cut-Band Removal

Removing

1. **DISCHARGE** the tag in the MyChild scanner.
2. **CUT** the band as shown. Position the tag in the scanner.
3. **CLEAN** the tag thoroughly in accordance with the manufacturer's instructions.
4. **PLACE** tag in foil pouch for storage.

2



[Click here to download the Cut Band Application Guide!](#)

Bracelet Tag Application, Removal, and Disinfection Guide

Bracelet Tag Application, Removal, and Disinfection Guide

Note: Applying a Bracelet Tag to a patient is done **after** the Tag's battery is tested and **after** the Tag has been assigned to the patient.

For more information on Tag battery testing, see *How to Store and Test a MyChild Security Tag*. For more information on assigning a Tag to a patient, see *Bracelet Tag Comprehensive Guide PowerPoint*.

Bracelet Tag Application

1. Feed the band through the two slots on the top of the tag (side with writing).

You may use any number of common bands.

2. Fasten the band around the limb to form a loop.
3. The Tag should be "just snug." Verify that the band and Tag do not restrict the patient's circulation or cause chafing.
4. Trim the excess band material if desired.
5. The Bracelet Tag can face either way. The benefit to having the top face up is you can view the Tag number.
6. Use Surgilast, if desired, to keep Tag in place.

Bracelet Tag Removal

Note: Remove the Tag **after** discharging the patient in the MyChild software.

To remove the tag, cut the band with scissor and slide the band out of the tag.

Bracelet Tag Disinfection Recommendations

- Thoroughly clean Tag with disinfectant wipe. McRoberts uses SaniZide Pro 1 disinfectant wipes which are broad-spectrum, EPA-registered, hospital-grade surface disinfecting wipes that kill the Novel Coronavirus (COVID-19) and meet OSHA (Occupational Safety and Health Administration) requirements regarding bloodborne pathogens and personal protection.
 - Use a small soft-bristled brush to scrub each tag with germicidal solution until there is no visible evidence of blood, body fluids, or material on the Tag. McRoberts uses SaniZide Pro 1, which is a broad-spectrum, EPA-registered, hospital-grade surface disinfectant that kills the Novel Coronavirus (COVID-19) and meets OSHA (Occupational Safety and Health Administration) requirements regarding bloodborne pathogens and personal protection.
 - Spray each Tag continuously with germicidal solution to ensure a minimum of 1 minute “wet time” with the solution.
 - Wipe dry with a disposable, lint-free towelette.
 - Spray Tags with alcohol (70%) to remove any chemical residue.
 - Wipe dry with a disposable, lint-free towelette. McRoberts uses Wypall towelettes.
- | | |
|--|--|
| | |
| | |

Tummy Tag Application, Removal, and Disinfection Guide



Applying a Tummy Tag to a
tested and *after* the Tag has
information on Tag battery
Security Tag. For more infor
Tummy Tag Comprehensive G

1. POSITION THE UMBILICA

- The MyChild Tummy
on umbilical cord cle



Tummy Tag and Disinfection

Tag Disinfection

- Thoroughly clean tag with disinfectant which are broad-spectrum, EPA-registered, effective against Novel Coronavirus (COVID-19) and meet requirements regarding bloodborne pathogens.
- Use a small soft-bristled brush to scrub evidence of blood, body fluids, or material with broad-spectrum, EPA-registered, hospital-grade disinfectant (COVID-19) and meets OSHA (Occupational Safety and Health) requirements regarding bloodborne pathogens and personal protective equipment.
- Spray each tag continuously with germicide solution with the solution.
- Wipe dry with a disposable, lint-free towel.
- Spray tags with alcohol (70%) to remove any remaining disinfectant.
- Wipe dry with a disposable, lint-free towel.

[Click here to download the Tummy Tag Application, Removal, and Disinfection Guide!](#)