

# Patient Functions

- [How To: Enroll a Patient](#)
- [How to: Unenroll a Patient](#)
- [How To: Use Bypass](#)
- [How To: Suspend Tamper](#)
- [How To: Locate Patient](#)

# How To: Enroll a Patient

Welcome, [admin@mychildcloud.com](mailto:admin@mychildcloud.com) | 🔔 10 | Show

Powered by McRoberts Technologies Visit us to

Home Patients **Click Patients Tab** Settings Admin Help Logout

**TAG SUMMARY**

Status	Count
InUse	6
Available	44
LowBattery	21
Bypass	1

Inventory Target: **N/A** Inventory Total: 72


**SUMMARY**

Status	Count
Active	6
Bypass	1



**UNACCEPTED ALARMS**

Tag near open door - 02/11/2022 20:11:32	Level 3
Tag near open door - 02/11/2022 20:09:32	Level 2
Tag near open door - 02/11/2022 18:44:03	Level 1
Tag near open door - 02/09/2022 21:04:40	
Tag near open door - 02/09/2022 21:04:35	
Tag near open door - 02/09/2022 21:04:31	
Tag near open door - 02/09/2022 21:04:26	
Tag near open door - 02/09/2022 21:04:21	
Tag near open door - 02/09/2022 21:04:18	

dev.mychildcloud.com:8080/Patients/Directory/


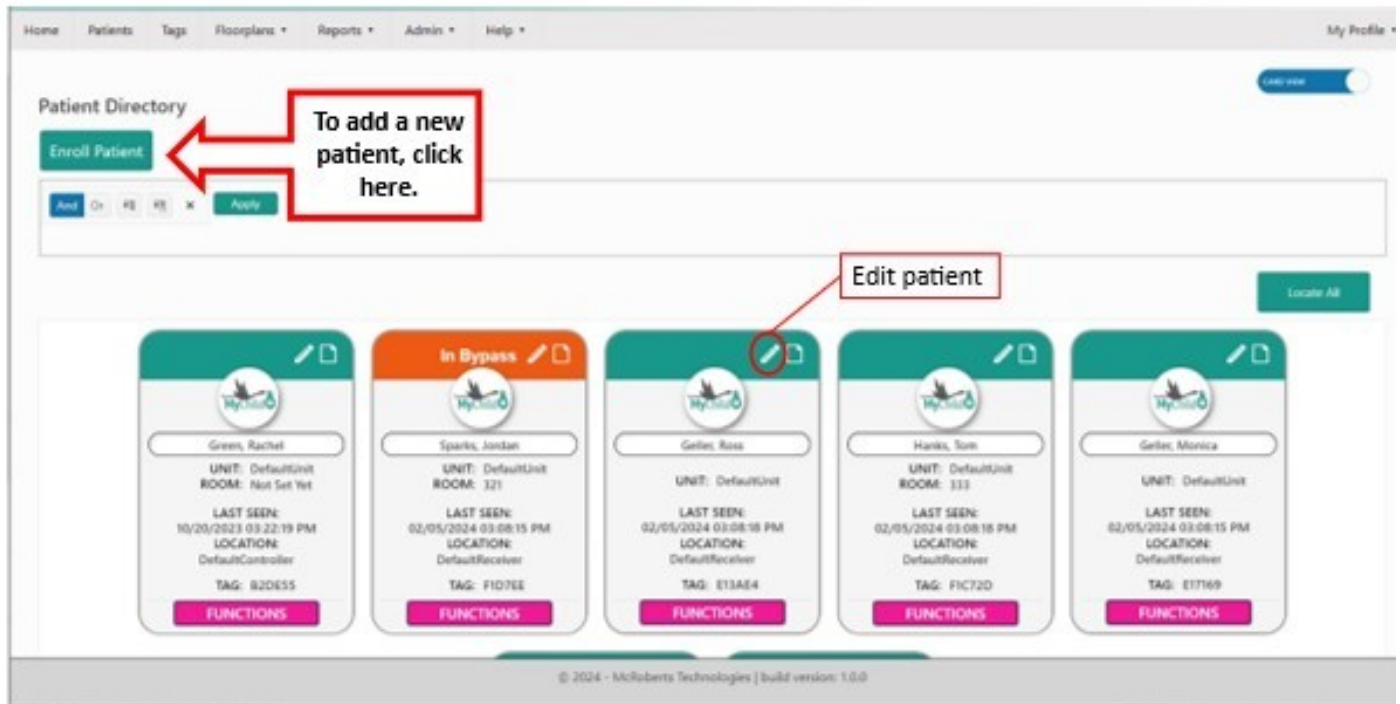



The enrollment window will pop up. Select the tag you wish to use from the drop-down list.

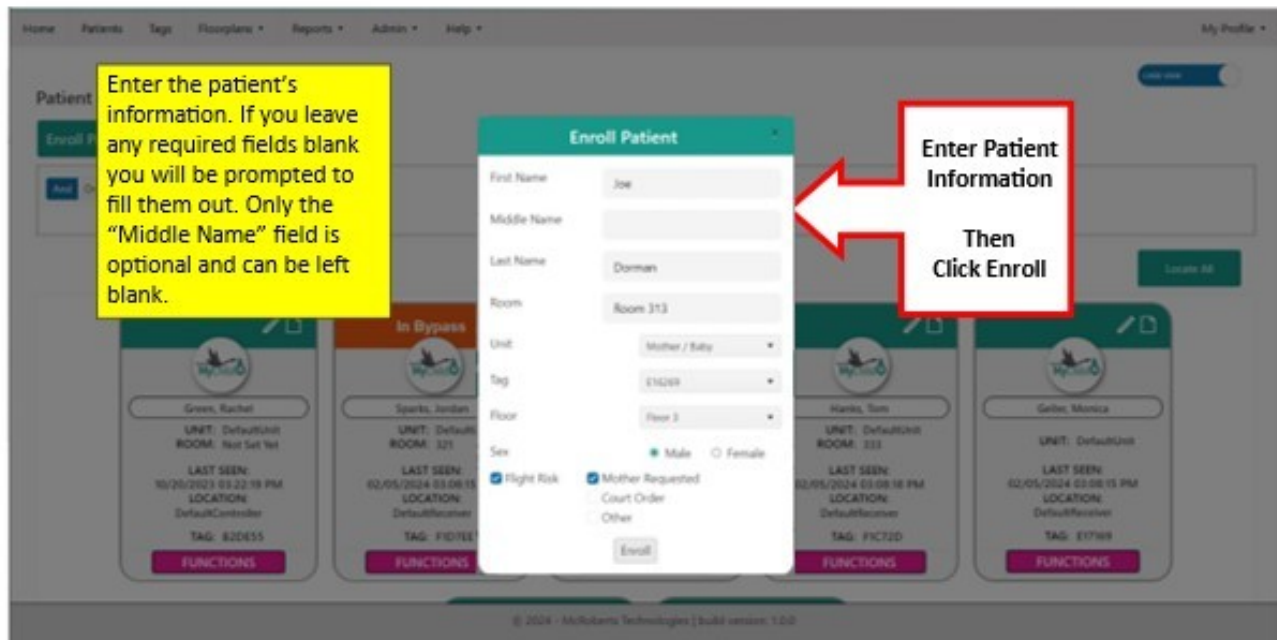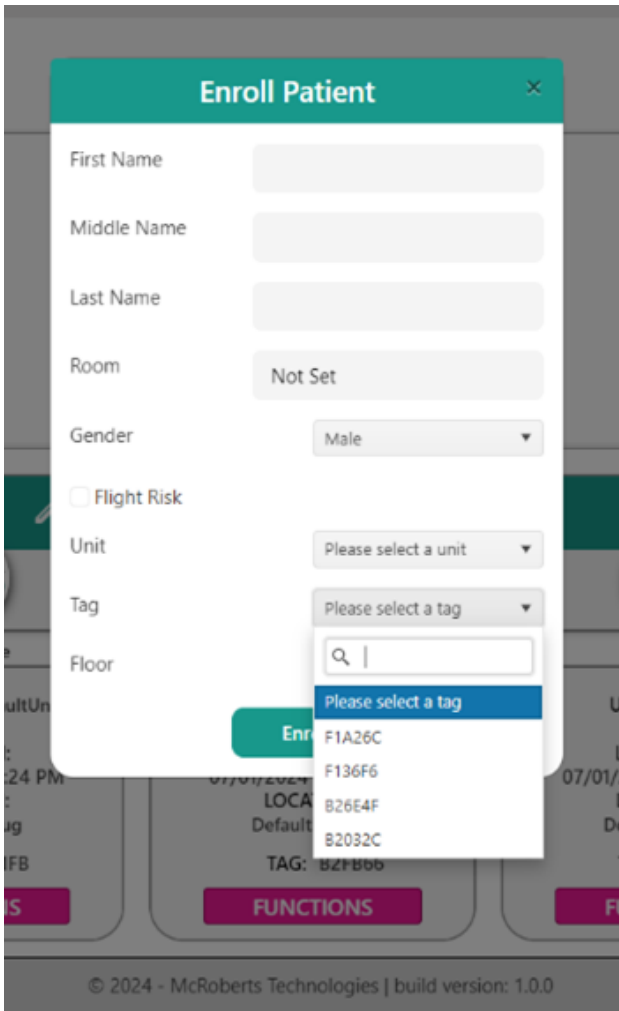

Patient will now have a status of "Active" and the total number of patients will be displayed on the dashboard. Other patient status are "Bypass" and "Tamper Suspended."

# How to: Unenroll a Patient

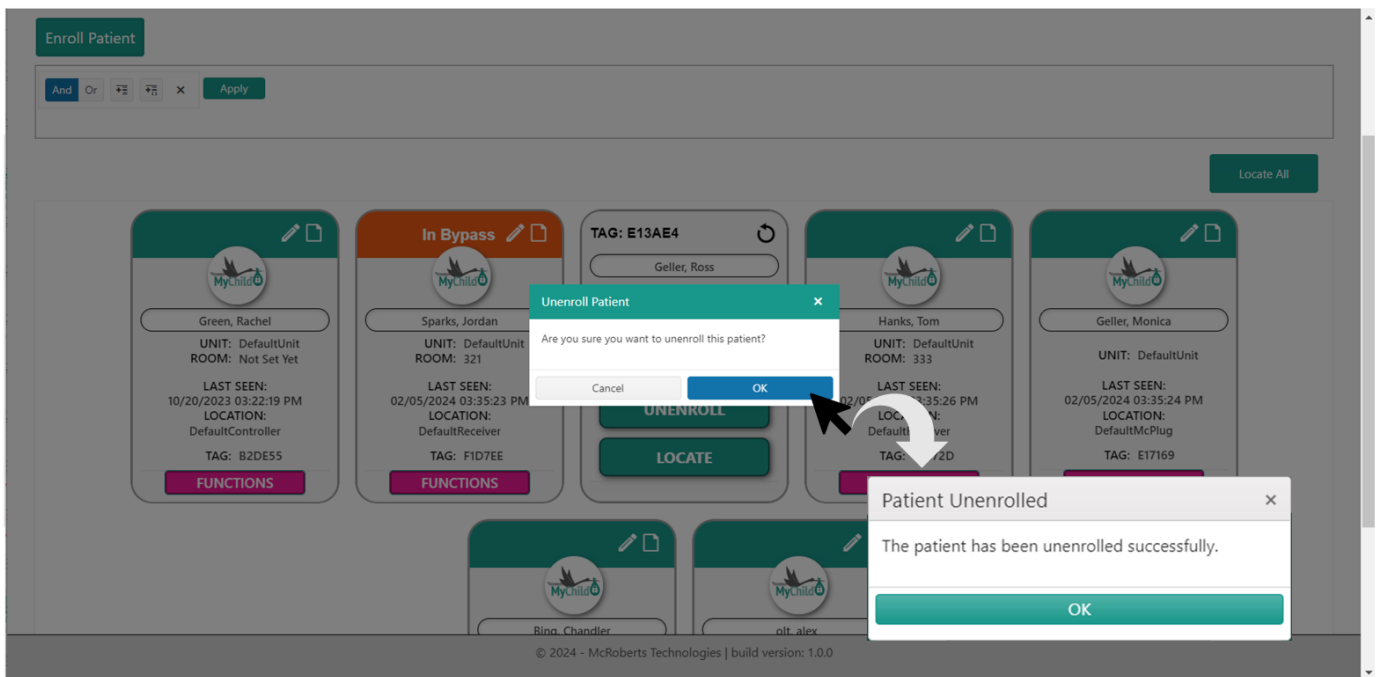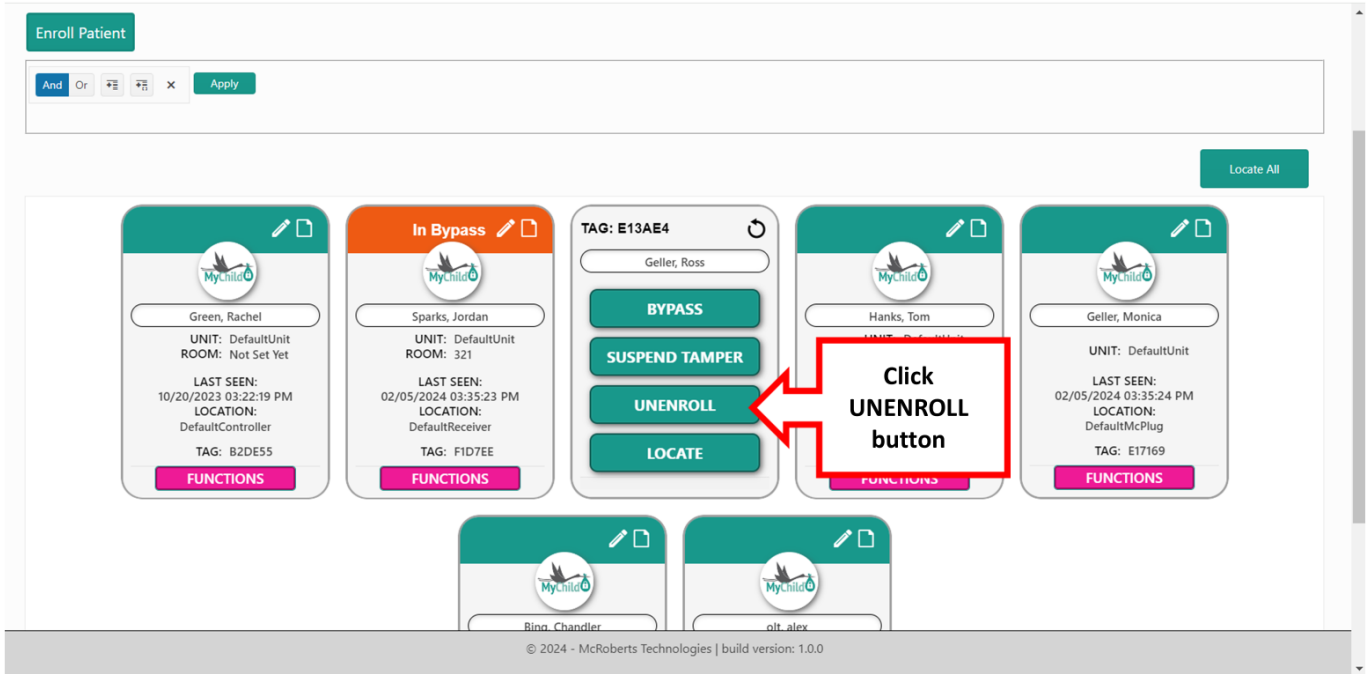


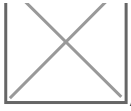

After clicking "OK" you will receive a notification that the patient has been unenrolled. The page will then refresh and the patient will be removed from the patient directory. The tag status will change to "available" meaning it can be used on a new patient.

# How To: Use Bypass

MyChild provides a ByPass that allows you to take a patient out of a protected area without setting off alarms. You must select a departing unit and a destination unit. These can be the same. The patient status changes from “Active” to “Bypass” when a patient is in ByPass.

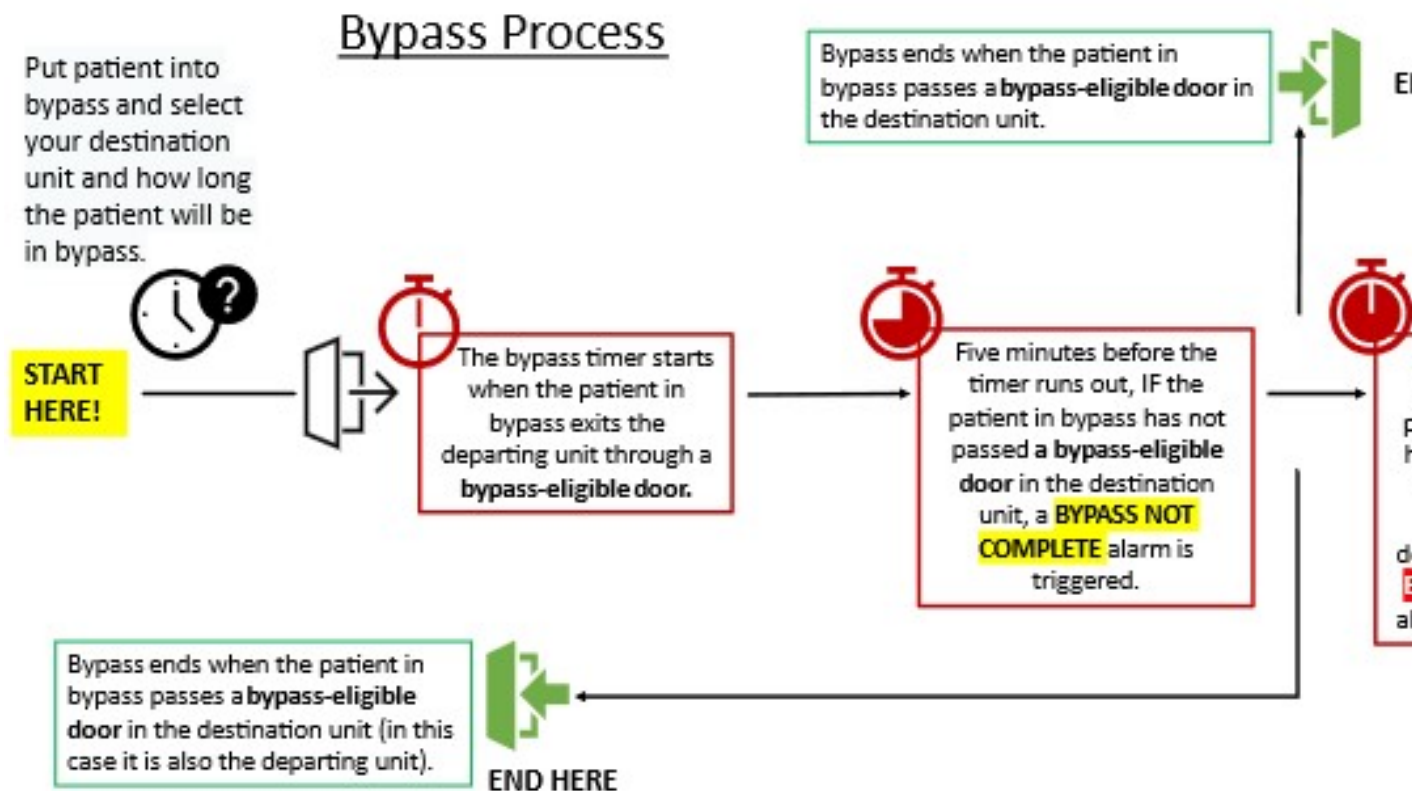



Enter a reason for the Bypass and a time limit.

Patient Bypass

Departing Unit: Labor and Delivery

Destination Unit: Mother / Baby

Time Required (in minutes): 5

Reason for Bypass: Moving

Transporter Name: All Smith (Optional)

Transporter Phone: (248) 555-9999 (Optional)

Put Patient In Bypass

Destination unit may be the same as the departing unit OR a different protected unit.

Enter in the time you will be out of protected units.

Bypass Mode starts when the tag passes a bypass-designated door in the departing unit. Bypass Mode ends when the tag in bypass passes through a designated door in the destination unit.

Once the information is complete, click the "Put Patient In Bypass" button. This will change the patient status to Bypass. You will also see this change on the dashboard under the Patient Summary.

A level one notification will appear on the upper right corner of the screen telling you the Bypass has started once the patient leaves through the first bypass designed door.

MyChild

Labor and Delivery - DefaultFloor

**BYPASS STARTED**

Full Name: Smith, Mike

Tag Number: E1F418

Assigned Room: 201

Location: West Door

Time Of Alarm: 02/27/2025 09:20:10 AM

Accept Alarm Edit Bypass

**Bypass Started**

Name: Rhodes, BG Casey

Assigned Room: Not Set

Tag Number: B2FB66

Alarm Time: 5/15/2024 2:37:49 PM

View Alarm

Level 3

Level 2

Level 1

Click "View Alarm" to view any Level 1 Alarm on the map.

This alarm indicates the patient has gone through the first door and the countdown has begun.

When the time allotted for the bypass is almost expired, a “BYPASS EXPIRING” alarm is triggered.

Home Patients Tags Floorplans Reports Admin Help Logout

### LABOR AND DELIVERY

McRoberts HOSPITAL

**BYPASS EXPIRING**

170 secs left

**Patient Name:** Davis, Brad  
**Tag Number:** B2DE55  
**Location Start:** Labor and Delivery  
**Location End:** Labor and Delivery  
**Transporter name:** Nurse Jackie  
**Transporter phone #:** 800-555-1234  
**Time of Alarm:** 05/03/2022 17:58:06

Accept Edit Bypass

Click “Accept” to clear the alarm from the screen. The timer will continue to run out until the bypass expires or is complete.

Click “Edit Bypass” to add more time to the bypass or edit the bypass details.

When the timer reaches zero, a “BYPASS EXPIRED” alarm is triggered. All bypass alarms (except BYPASS COMPLETE) allow you to edit the bypass. Changes can also be made on the patient card.

MyChild MyChild

Home Patients Tags Floorplans Reports Admin Help Logout

### LABOR AND DELIVERY

McRoberts HOSPITAL

**BYPASS EXPIRED**

<b>Patient Name:</b>	Davis, Brad
<b>Tag Number:</b>	B2DE55
<b>Location Start:</b>	Labor and Delivery
<b>Location End:</b>	Labor and Delivery
<b>Transporter name:</b>	Nurse Jackie
<b>Transporter phone #:</b>	800-555-1234
<b>Time of Alarm:</b>	05/03/2022 17:55:36


Accept Edit Bypass

RN needs more time, edit bypass required

© 2022 - McRoberts Technologies | built

First, click the "Accept" button, then enter a message and click the green arrow to accept the alarm. The Bypass Expired alarm only happens if the time limit has run out and the patient has not passed through the second bypass designated door. If the patient arrives on time, you will receive a Bypass Complete alarm. This will show up in the upper right corner as a notification as it is a level one alarm. Click on the notification to view the alarm detail.

## LABOR AND DELIVERY

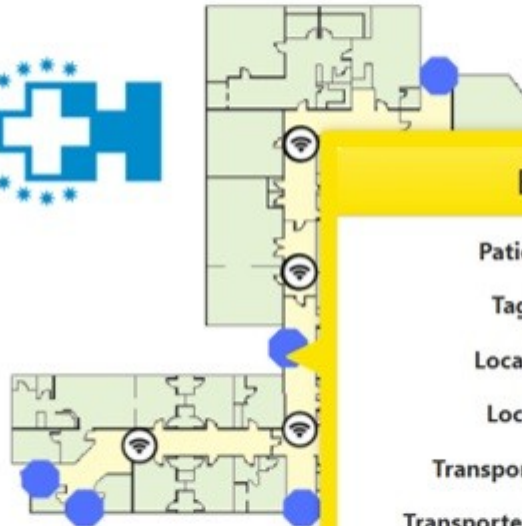

Labor and Delivery

## BYPASS COMPLETE

**Patient Name:** Davis, Brad  
**Tag Number:** B2DE55  
**Location Start:** Labor and Delivery  
**Location End:** Labor and Delivery  
**Transporter name:** Nurse Jackie  
**Transporter phone #:** 800-555-1234  
**Time of Alarm:** 05/03/2022 17:44:29

When you complete a bypass by clicking “Complete,” you are acknowledging that the patient has arrived at the destination unit. The patient status will return to "Active".

For additional help, call 1-800-776-7328 and press 2 for technical support.

[Click here to download PDF.](#)



# How To: Suspend Tamper

Suspend Tamper Alarms: MyChild p  
remove a cut-band or sensing tag tem  
into SUSPEND TAMPER mode, the pat

The screenshot displays the 'Enroll Patient' interface. At the top, there is a green header with the text 'Enroll Patient'. Below this, there are filter options: 'And', 'Or', and two icons. A red-bordered box contains the text: 'Click **FUNCTIONS** to see all patient functions.' Below the filter options, there are two patient cards. The first card has a green header and displays the following information: 'Green, Rachel', 'UNIT: DefaultUnit', 'ROOM: Not Set Yet', 'LAST SEEN: 10/20/2023 03:22:19 PM', 'LOCATION: DefaultController', and 'TAG: B2DE55'. A pink 'FUNCTIONS' button is at the bottom of this card. A blue curved arrow points from the 'FUNCTIONS' button of the first card to the second card. The second card has an orange header and displays: 'In Bypass', 'TAG: F1D7EE', and a pink 'FUNCTIONS' button. A red-bordered box on the second card contains the text: 'Click **SUSPEND TAMPER** button.'

When the time allotted for the susper  
EXPIRING” alarm is triggered. When t  
is triggered. All suspend tamper alarm  
be made on the patient card.

When you click Complete to complet  
back on the patient.

Alarm Name	Level	Alarm Description
SUSPEND TAMPER STARTED	1	This alarm notifies a patient has been Suspend Tamper that the Suspend timer has started  The patient status from “Active” to “Suspended.”
SUSPEND	1	This alarm appears

## Suspend Tamper PDF

*When you click “Complete” to complete a Suspend Tamper, you are acknowledging that the tag is back on the patient.*

# How To: Locate Patient