

How To: Suspend Tamper

Suspend Tamper Alarms: MyChild p
remove a cut-band or sensing tag tem
into SUSPEND TAMPER mode, the pat

Enroll Patient

And Or

Click **FUNCTIONS** to see all patient functions.

MyChild

Green, Rachel

UNIT: DefaultUnit
ROOM: Not Set Yet

LAST SEEN:
10/20/2023 03:22:19 PM
LOCATION:
DefaultController

TAG: B2DE55

FUNCTIONS

In Bypass

Click **SUSPEND TAMPER** button.

TAG: F1D7EE

FUNCTIONS

When the time allotted for the susper
EXPIRING” alarm is triggered. When t
is triggered. All suspend tamper alarm
be made on the patient card.

When you click Complete to complet
back on the patient.

Alarm Name	Level	Alarm Description
SUSPEND TAMPER STARTED	1	This alarm notifies a patient has been Suspend Tamper that the Suspend timer has started The patient status from “Active” to “Suspended.”
SUSPEND	1	This alarm appears

Suspend Tamper PDF

When you click “Complete” to complete a Suspend Tamper, you are acknowledging that the tag is back on the patient.

Revision #8

Created 1 February 2022 16:55:21 by Admin

Updated 5 February 2026 19:07:26 by Anna Olt