

How To: Suspend Tamper

MyChild provides a suspend tamper function which allows you to remove a tag temporarily without setting off alarms. To remove a cut-band or sensing tag temporarily, put the patient in SUSPEND TAMPER. The patient status changes from “Active” to “Tamper Suspended.”

The screenshot displays the MyChild Patient Directory interface. At the top is a teal header with the MyChild logo and a navigation bar containing links: Home, Patients, Tags, Floorplans, Reports, Admin, Help, and Logout. Below the header, the section is titled "Patient Directory".

The directory contains several patient cards. The first card on the left is a teal card with a white circle containing a plus sign and the word "NEW". Below this is a white box with the MyChild logo and the text "SECURITY FIRST".

The second card is for "Davis, Brad". It features the MyChild logo, a pencil icon, and a document icon. The patient details are as follows:

Field	Value
SEX:	MALE
UNIT:	PEDS
TAG:	D1F5A2

At the bottom of the card is a pink button labeled "FUNCTIONS" with a circular arrow icon.

The third card is for "Olt, Anna". It also features the MyChild logo, a pencil icon, and a document icon. The patient details are as follows:

Field	Value
SEX:	FEMALE
UNIT:	PEDS
TAG:	E150CD

At the bottom of the card is a pink button labeled "FUNCTIONS" with a circular arrow icon. Below this button is a yellow bar with the text "FLIGHT RISK".

Below these cards, the top of two more cards are visible. The first is for "Davis, Sarah", showing the MyChild logo, a pencil icon, a document icon, and the name "Davis, Sarah". The second card is partially visible on the right, showing the MyChild logo and the name "Be".

Complete Suspend Tamper details:

The image shows a mobile application interface. On the left, a 'Patient Suspend Tamper' dialog box is open, featuring a close button (X) in the top right corner. It contains three input fields: 'Time Required (in minutes):' with a numeric keypad, 'Reason for Suspend:', and 'Nurse's Name:'. A 'Create' button is at the bottom of the dialog. Behind the dialog, a patient card is visible, showing 'FEMALE', 'PEDS', and 'F1D71B'. Below the card are two buttons: 'FUNCTIONS' with a refresh icon and a yellow 'FLIGHT RISK' banner. On the right, another patient card for 'Woods, Gloria' is shown with a 'NO PHOTO' placeholder. It has four buttons: 'BYPASS', 'SUSPEND TAMPER' (highlighted with a yellow box), 'UNENROLL', and 'LOCATE'.

Once the information is complete, click the Create button.
This will change the patient status to “Tamper Suspended.”

When the time allotted for the suspend tamper is almost expired, a “SUSPEND TAMPER EXPIRING” alarm is triggered. When the timer reaches zero, a “SUSPEND TAMPER EXPIRED” alarm is triggered. All suspend tamper alarms allow you to edit the suspend tamper. Changes can also be made on the patient card.

When you click “Complete” to complete a Suspend Tamper, you are acknowledging that the tag is back on the patient.

Revision #6

Created 1 February 2022 16:55:21 by Admin

Updated 25 July 2022 14:53:14 by Anna Olt