


How To: Suspend Tamper

MyChild provides a suspend tamper function which allows you to remove a tag temporarily without setting off alarms. To remove a cut-band or sensing tag temporarily, put the patient in SUSPEND TAMPER. The patient status changes from “Active” to “Tamper Suspended.”

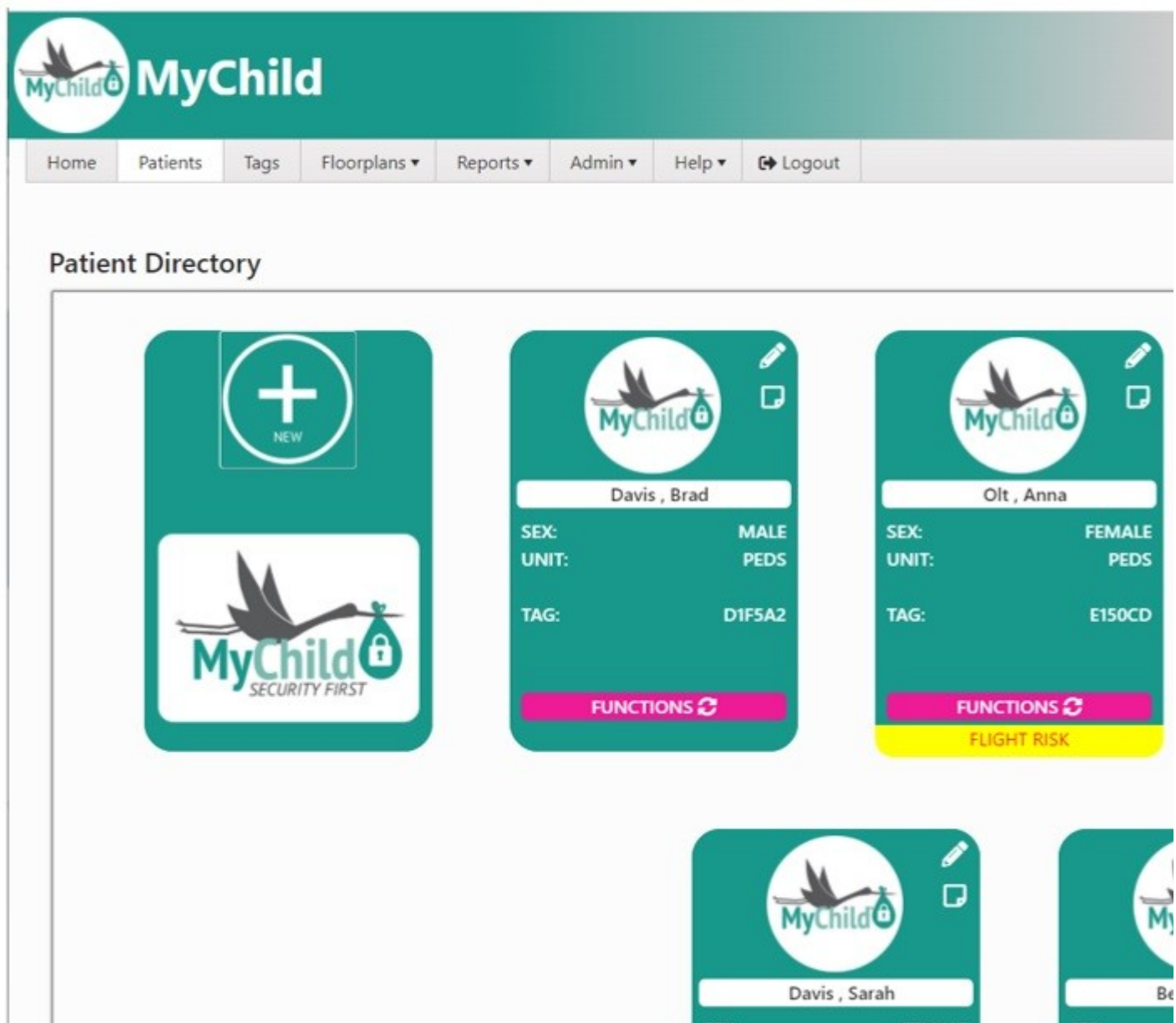

Complete Suspend Tamper details:

The image shows a mobile application interface. On the left, a 'Patient Suspend Tamper' dialog box is open over a patient card. The dialog box contains the following fields: 'Time Required (in minutes):' with a numeric input field and up/down arrows; 'Reason for Suspend:' with a text input field; and 'Nurse's Name:' with a text input field. A 'Create' button is at the bottom of the dialog. The patient card in the background shows 'FEMALE PEDS' and 'F1D71B'. Below the card are two buttons: 'FUNCTIONS' with a refresh icon and 'FLIGHT RISK' in a yellow box. On the right, another patient card for 'Woods, Gloria' is shown. It has a 'NO PHOTO' placeholder. Below the name are four buttons: 'BYPASS', 'SUSPEND TAMPER' (highlighted with a yellow box), 'UNENROLL', and 'LOCATE'.

Once the information is complete, click the Create button.
This will change the patient status to “Tamper Suspended.”

When the time allotted for the suspend tamper is almost expired, a “SUSPEND TAMPER EXPIRING” alarm is triggered. When the timer reaches zero, a “SUSPEND TAMPER EXPIRED” alarm is triggered. All suspend tamper alarms allow you to edit the suspend tamper. Changes can also be made on the patient card.

When you click “Complete” to complete a Suspend Tamper, you are acknowledging that the tag is back on the patient.

Revision #6

Created 1 February 2022 16:55:21 by Admin

Updated 25 July 2022 14:53:14 by Anna Olt