


# How To: Use Bypass Mode

MyChild provides a ByPass Mode that allows you to take a patient out of a protected area without setting off alarms. You must select a departing unit and a destination unit. These can be the same. The patient status changes from “Active” to “Bypass” when a patient is in ByPass Mode.

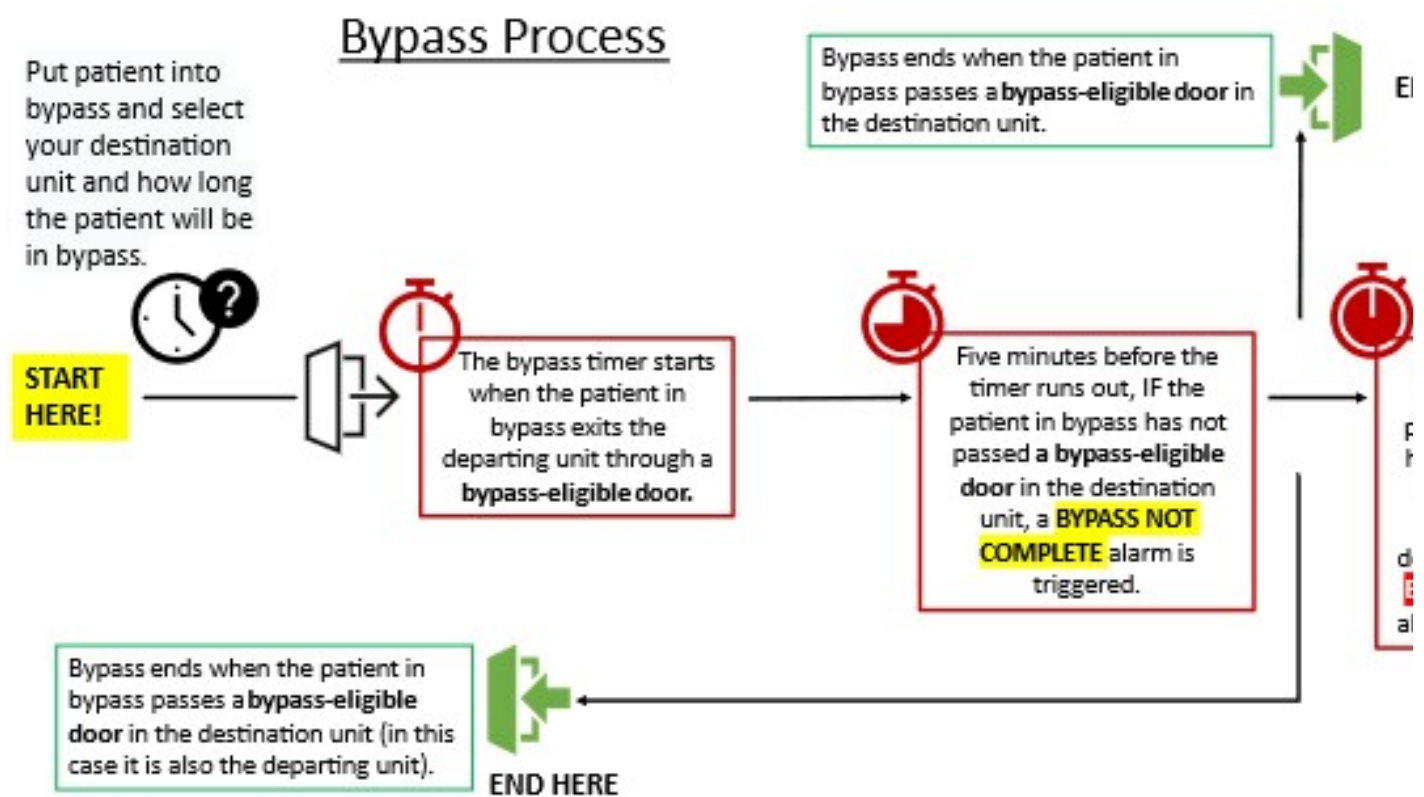

Home Patients Tags Floorplans Reports Admin Help My Profile

Patient Directory

Enroll Patient

And Or Add X Apply

Locate All

Access all patient functions by clicking the pink **FUNCTIONS** button on the card.

Low Battery

Granger, Hermione

UNIT: Mother / Baby  
ROOM: 310

LAST SEEN:  
02/21/2024 11:07:21 AM  
LOCATION:  
Room 311  
TAG: 821680

FUNCTIONS

Angelo-Heimer-Schmidt, John

UNIT: Mother / Baby  
ROOM: 311

LAST SEEN:  
02/21/2024 11:07:21 AM  
LOCATION:  
Room 311  
TAG: 821680

FUNCTIONS

TAG: E2484F

Rhodes, Daniel

BYPASS

SUSPEND TAMPER

UNENROLL

LOCATE

Dorman, Joe

UNIT: Mother / Baby  
ROOM: 313

LAST SEEN:  
02/21/2024 11:25:10 AM  
LOCATION:  
Room 313  
TAG: E16269

FUNCTIONS

FLIGHT RISK

Low Battery

Doe, John

UNIT: Mother / Baby  
ROOM: 315

LAST SEEN:  
02/21/2024 11:05:41 AM  
LOCATION:  
Room 315  
TAG: E10488

FUNCTIONS

Home Patients Tags Floorplans Reports Admin Help My Profile

Patient Directory

Enroll Patient

And Or Add X Apply

Locate All

Click Bypass to continue

Low Battery

Granger, Hermione

UNIT: Mother / Baby  
ROOM: 310

LAST SEEN:  
02/21/2024 11:05:41 AM  
LOCATION:  
Room 310  
TAG: 821580

FUNCTIONS

Angelo-Heimer-Schmidt, John

UNIT: Mother / Baby  
ROOM: 311

LAST SEEN:  
02/21/2024 11:07:21 AM  
LOCATION:  
Room 311  
TAG: 821680

FUNCTIONS

TAG: E2484F

Rhodes, Daniel

BYPASS

SUSPEND TAMPER

UNENROLL

LOCATE

Low Battery


Doe, John

UNIT: Mother / Baby  
ROOM: 315

LAST SEEN:  
02/21/2024 11:05:41 AM  
LOCATION:  
Room 315  
TAG: E10488

FUNCTIONS

Enter a reason for the Bypass and a time limit.

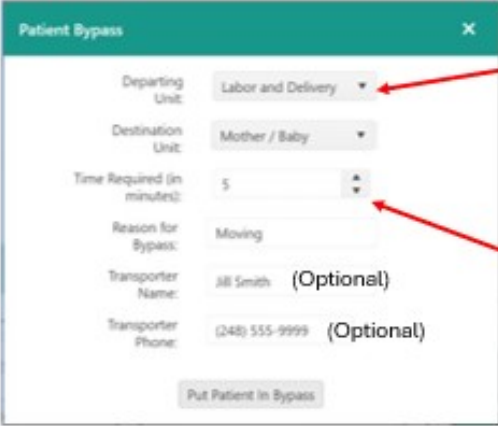

The screenshot shows a 'Patient Bypass' form with the following fields:

- Departing Unit: Labor and Delivery
- Destination Unit: Mother / Baby
- Time Required (in minutes): 5
- Reason for Bypass: Moving
- Transporter Name: Jill Smith (Optional)
- Transporter Phone: (248) 555-9999 (Optional)

At the bottom is a button labeled 'Put Patient In Bypass'.

Two red callout boxes provide additional information:

- The first callout points to the 'Destination Unit' dropdown and states: "Destination unit may be the same as the departing unit OR a different protected unit."
- The second callout points to the 'Time Required' field and states: "Enter in the time you will be outside the protected units."

Below the second callout, a paragraph explains: "Bypass Mode starts when the tag in bypass passes a bypass-designated door in the departing unit. Bypass Mode ends when the tag in bypass passes through a bypass-designated door in the destination unit."

Once the information is complete, click the "Put Patient In Bypass" button. This will change the patient status to Bypass. You will also see this change on the dashboard under the Patient Summary.

A level one notification will appear on the upper right corner of the screen telling you the Bypass has started once the patient leaves though the first bypass designed door.

MyChild

MyChild

Home

Patients

Tags

Floorplans ▾

Reports ▾


Admin ▾

Help ▾

Logout

LABOR AND DELIVERY

McRoberts HOSPITAL

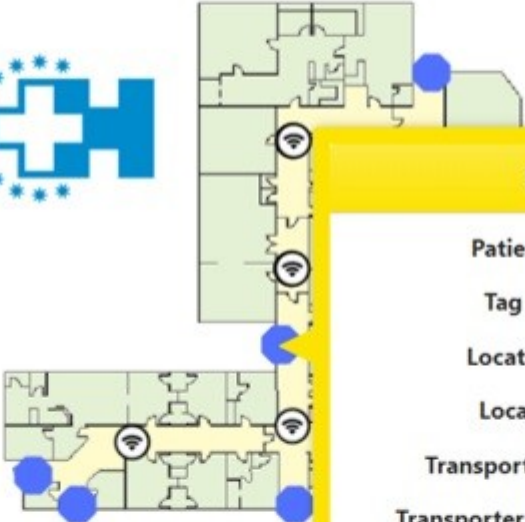

Labor and Delivery

BYPASS STARTED

Patient Name:

Davis, Brad

Tag Number:

B2DE55

Location Start:

Labor and Delivery

Location End:

Labor and Delivery

Transporter name:

Nurse Jackie

Transporter phone #:

800-555-1234

Time of Alarm:

05/03/2022 17:43:09

Accept



Edit Bypass

© 2022 - McRoberts Technologies | build version: 8122

When the time allotted for the bypass is almost expired, a “BYPASS EXPIRING” alarm is triggered.

Home Patients Tags Floorplans ▼ Reports ▼ Admin ▼ Help ▼ Logout

## LABOR AND DELIVERY

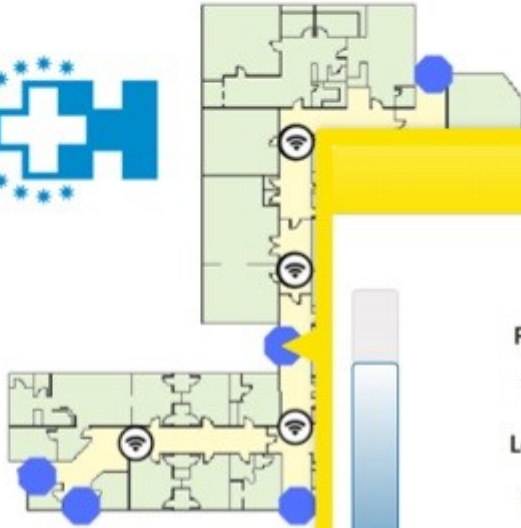

Labor and Delivery

### BYPASS EXPIRING

170 secs left

**Patient Name:** Davis, Brad  
**Tag Number:** B2DE55  
**Location Start:** Labor and Delivery  
**Location End:** Labor and Delivery  
**Transporter name:** Nurse Jackie  
**Transporter phone #:** 800-555-1234  
**Time of Alarm:** 05/03/2022 17:58:06


[Accept](#) [Edit Bypass](#)

Click “Accept” to clear the alarm from the screen. The timer will continue to run out until the bypass expires or is complete.

Click “Edit Bypass” to add more time to the bypass or edit the bypass details.

When the timer reaches zero, a “BYPASS EXPIRED” alarm is triggered. All bypass alarms (except BYPASS COMPLETE) allow you to edit the bypass. Changes can also be made on the patient card.



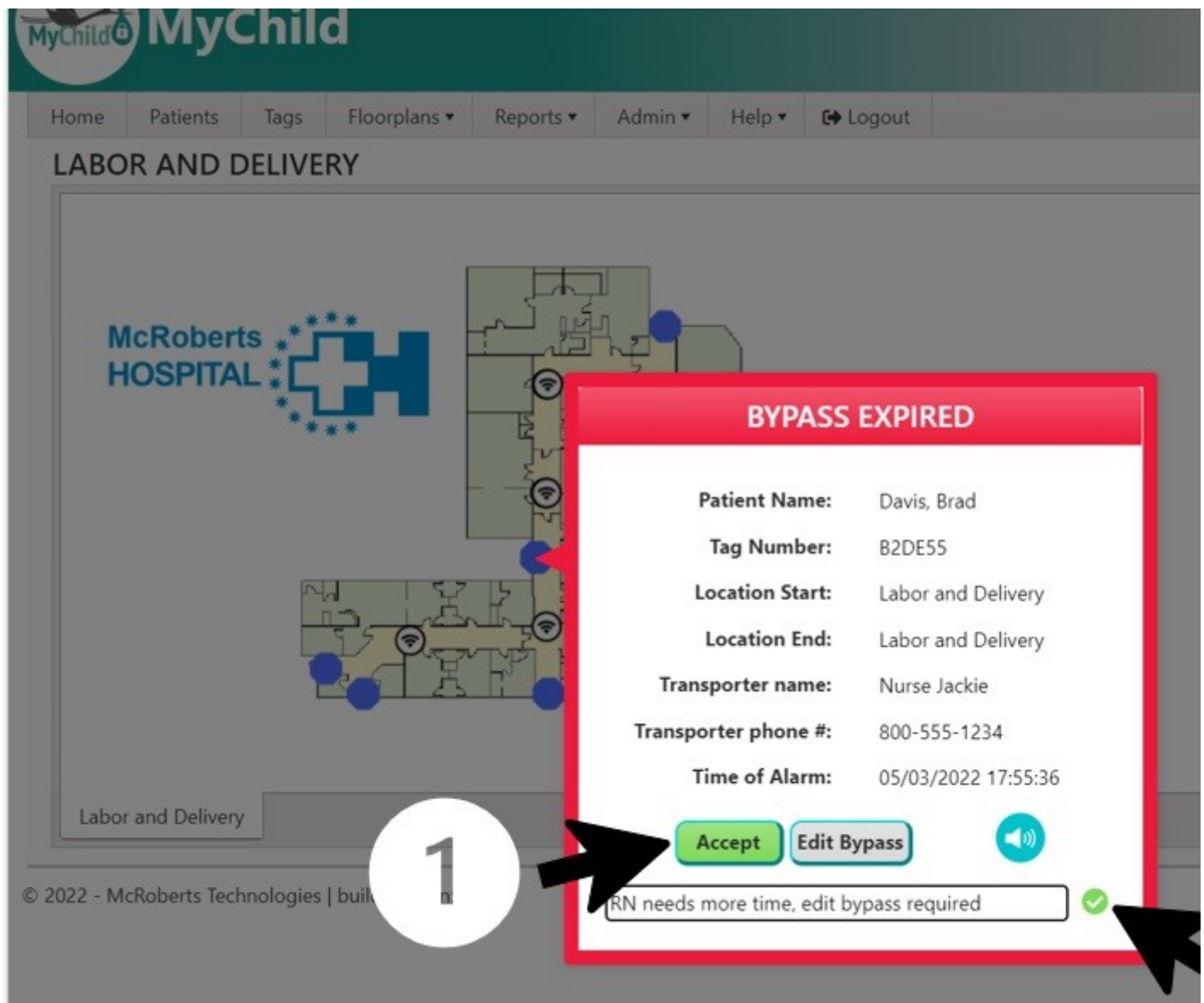


First, click the "Accept" button, then enter a message and click the green arrow to accept the alarm. The Bypass Expired alarm only happens if the time limit has run out and the patient has not passed through the second bypass designated door. If the patient arrives on time, you will receive a Bypass Complete alarm. This will show up in the upper right corner as a notification as it is a level one alarm. Click on the notification to view the alarm detail.



## LABOR AND DELIVERY

McRoberts  
HOSPITAL

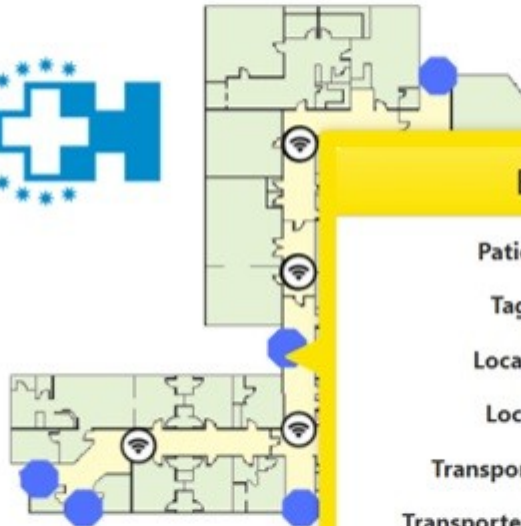

Labor and Delivery

### BYPASS COMPLETE

<b>Patient Name:</b>	Davis, Brad
<b>Tag Number:</b>	B2DE55
<b>Location Start:</b>	Labor and Delivery
<b>Location End:</b>	Labor and Delivery
<b>Transporter name:</b>	Nurse Jackie
<b>Transporter phone #:</b>	800-555-1234
<b>Time of Alarm:</b>	05/03/2022 17:44:29

Accept

When you complete a bypass by clicking “Complete,” you are acknowledging that the patient has arrived at the destination unit. The patient status will return to "Active".

For additional help, call 1-800-776-7328 and press 2 for technical support.

[Click here to download PDF.](#)

---

Revision #20

Created 1 February 2022 16:55:04 by Admin

Updated 4 April 2025 18:49:27 by Anna Olt