


Patient Alarms

Patient Alarms are alarms that are triggered by assigned tags under certain conditions. Patient alarms show on your floorplan and point to the device that triggered that alarm or saw the tag last. The alarm includes the patient's name, tag number, and last known location.

Level One Patient Alarms:

Level 1 alarms notify you about events that could lead to a security breach. Level 1 alarms are indicated by pop-up notifications that disappear after 30 seconds unless you click "View Alarm." The alarm is displayed in the floorplan view. The floorplan view has priority over all other pages. Make sure you complete whatever you are working on before clicking on "View Alarm" because your work will not be saved.

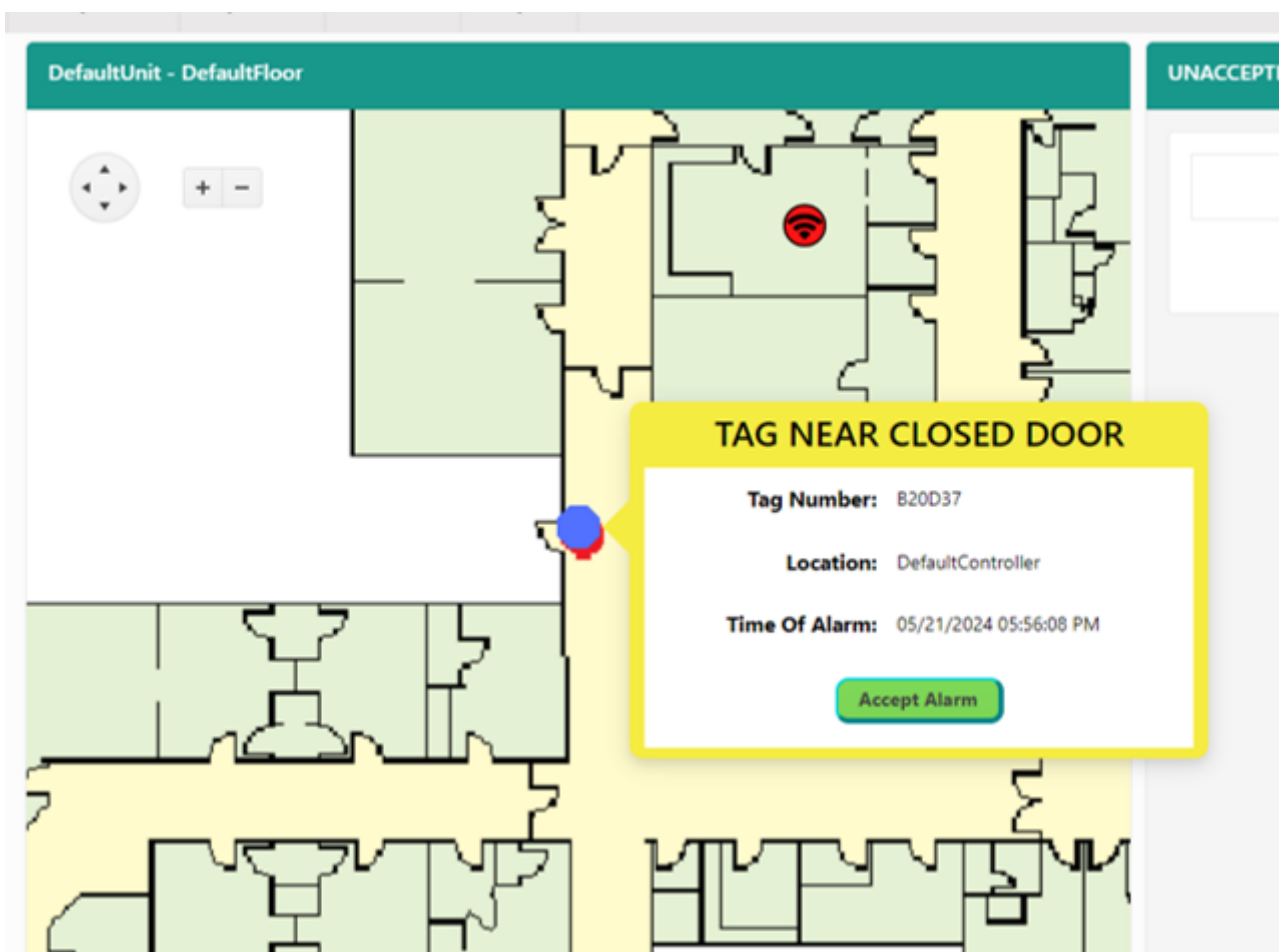

TAG NEAR CLOSED DOOR

Level Two Patient Alarms

Level 2 alarms also notify you about events that could lead to a security breach but are more serious than the events that trigger a level 1 alarm. It is recommended that security protocols be followed. There are no pop-up notifications. The floorplan view with the alarm automatically takes over your screen; therefore, whatever you are working on will not be saved. Audio is included.

TAG LOITERING NEAR DOOR: This alarm occurs when an assigned tag that is NOT in Bypass Mode is in a door field for longer than one minute and the door is closed. It replaces the TAG NEAR CLOSED DOOR alarm.

LOW BATTERY TAG ON PATIENT: This alarm occurs when the system receives a low battery message from a tag that is assigned to a patient.

MyChild

Home Patients Tags Floorplans ▼ Reports ▼ Admin ▼ Help ▼ Logout

LABOR AND DELIVERY

McRoberts HOSPITAL

LOW BATTERY TAG ON PATIENT

Change Tag

Device Name: East Hall Nurses Stati

Patient Name: Olt, Alex

Assigned Room:

Tag Number: F1E753

Time of Alarm: 03/28/2022 14:03:57

ACCEPT

tag changed

First, click "ACCEPT" button.

1

Labor and Delivery


© 2022 - McRoberts Technologies | build version: 8122

PATIENT UNAUTHORIZED BY MYCHILD

Level Three Patient Alarms:

TAG TAMPER

TAG NEAR OPEN DOOR

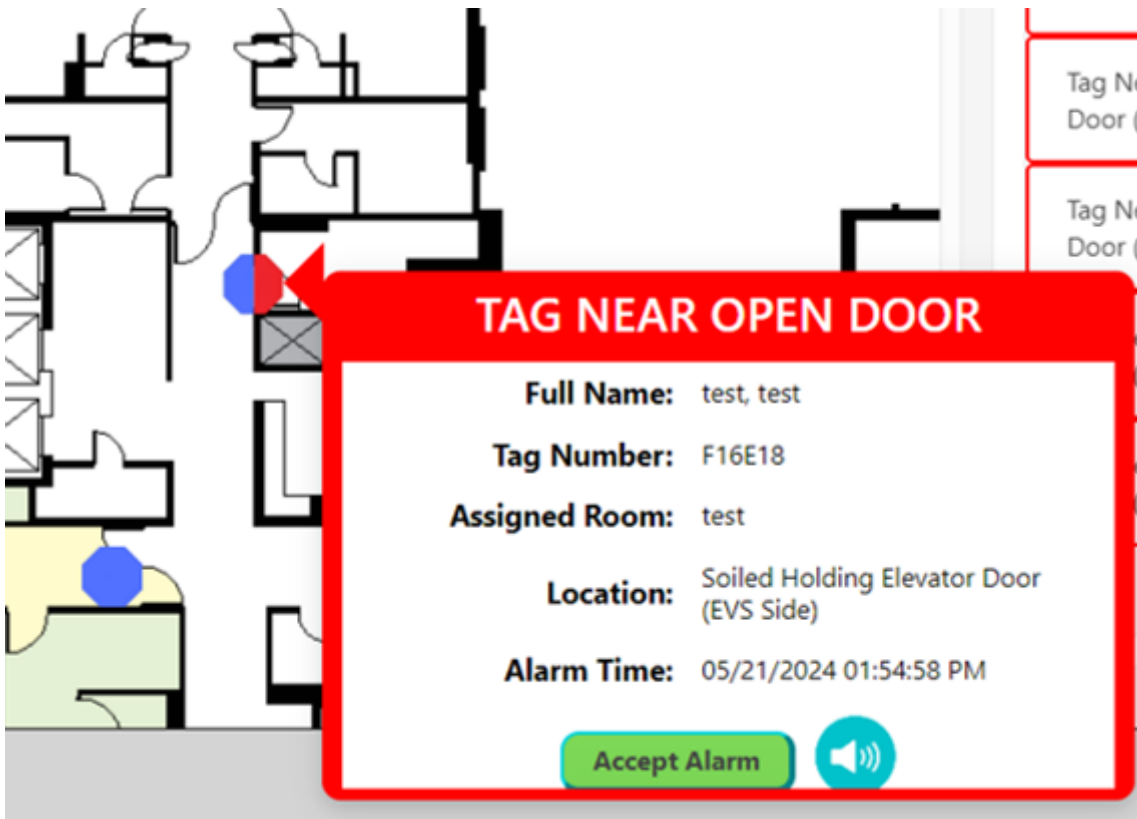

TAG NEAR ELEVATORS DOORS CLOSED

TAG NEAR ELEVATORS DOOR OPEN

TAG IN ELEVATOR

TAG OUT OF PROTECTED AREA

Revision #6

Created 16 February 2022 16:39:28 by Admin

Updated 25 July 2024 18:14:02 by Anna Olt