

Tummy Tag Application, Removal, and Disinfection Guide



Applying a Tummy Tag to a
tested and *after* the Tag has
information on Tag battery
Security Tag. For more infor
Tummy Tag Comprehensive G

1. POSITION THE UMBILICA

- The MyChild Tummy
on umbilical cord cle



Tummy Tag and Disinfection

Tag Disinfection

- Thoroughly clean tag with disinfectant which are broad-spectrum, EPA-registered, effective against Novel Coronavirus (COVID-19) and meet requirements regarding bloodborne pathogens.
- Use a small soft-bristled brush to scrub evidence of blood, body fluids, or material with broad-spectrum, EPA-registered, hospital-grade disinfectant (COVID-19) and meets OSHA (Occupational Safety and Health) requirements regarding bloodborne pathogens and personal protective equipment.
- Spray each tag continuously with germicide solution with the solution.
- Wipe dry with a disposable, lint-free towel.
- Spray tags with alcohol (70%) to remove any remaining disinfectant.
- Wipe dry with a disposable, lint-free towel.

[Click here to download the Tummy Tag Application, Removal, and Disinfection Guide!](#)

Revision #9

Created 4 February 2022 16:35:35 by Admin

Updated 19 January 2023 21:32:27 by Anna Olt