

# What is Bypass?

MyChild provides a ByPass Mode that allows you to take a patient out of a protected area without setting off alarms. You must select a departing unit and a destination unit. These can be the same. The patient status changes from “Active” to “Bypass” when a patient is in ByPass Mode.

## The Bypass Process

Put patient into Bypass Mode and select how much time you want the Bypass Mode to be in effect.



Bypass time begins when you pass the **first bypass-designated door**

Bypass Mode ends when you pass the **first bypass-designated door a second time.**





Click  
Patients  
Tab

### TAG SUMMARY

| Status     | Count |
|------------|-------|
| InUse      | 6     |
| Available  | 44    |
| LowBattery | 21    |
| Bypass     | 1     |

Inventory Target: **N/A**      Inventory Total: 72

### SUMMARY

| Status | Count |
|--------|-------|
| Active | 6     |
| Bypass | 1     |

### UNACCEPTED ALARMS

- Tag near open d


### Patient Directory

A teal patient card for Brad Davis. It features the MyChild logo at the top, a name bar with "Davis, Brad", and fields for "SEX: MALE", "UNIT: DefaultUnit", and "TAG: D1F5A2". A pink button at the bottom is labeled "FUNCTIONS" with a refresh icon.A teal patient card for Anna Olt. It features the MyChild logo at the top, a name bar with "Olt, Anna", and fields for "SEX: FEMALE" and "UNIT: DefaultUnit".

Access all patient functions by clicking the pink functions button on the patient card.

A teal patient card for Sarah Davis. It features the MyChild logo at the top, a name bar with "Davis, Sarah", and fields for "SEX: FEMALE" and "UNIT: DefaultUnit".A teal patient card for Tag, Test. It features the MyChild logo at the top, a name bar with "Tag, Test", and fields for "SEX: MALE" and "UNIT: DefaultUnit".

## Patient Directory

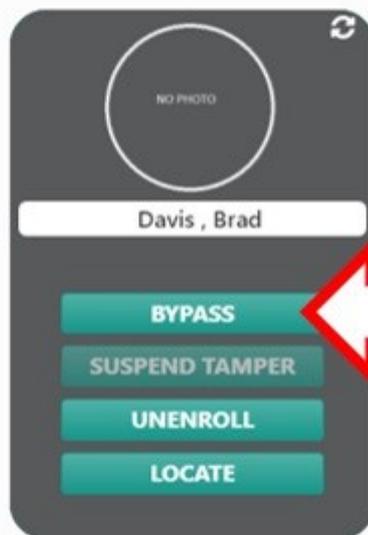

Click Bypass to continue

Enter a reason for the Bypass, as well as your name

The image shows two parts of a mobile application interface. On the left is a patient card for 'Davis, Brad' with a 'NO PHOTO' placeholder and a yellow-bordered 'BYPASS' button. On the right is a 'Patient Bypass' form with fields for 'Departing Unit' (PEDS), 'Destination Unit' (Select a department...), 'Time Required (in minutes)', 'Reason for Bypass', 'Transporter Name', and 'Transporter Phone'. A red arrow points to the 'Time Required' field. A 'Create' button is at the bottom.

Once the information is complete, click the Create button. This will change the patient status to "Bypass."

A level one notification will appear on the upper right corner of the screen telling you the Bypass has started once the patient leaves though the first bypass designed door.

The screenshot displays the MyChild software interface. At the top, there is a navigation bar with the following items: Home, Patients, Tags, Floorplans (with a dropdown arrow), Reports (with a dropdown arrow), Admin (with a dropdown arrow), Help (with a dropdown arrow), and Logout (with a double arrow icon). Below the navigation bar, the main heading is "LABOR AND DELIVERY".

The central part of the interface shows a floorplan of the Labor and Delivery department. The floorplan is a green and white diagram with several blue circular markers indicating tag locations. A yellow callout box is overlaid on the floorplan, containing the following information:

**BYPASS STARTED**


|                             |                     |
|-----------------------------|---------------------|
| <b>Patient Name:</b>        | Davis, Brad         |
| <b>Tag Number:</b>          | B2DE55              |
| <b>Location Start:</b>      | Labor and Delivery  |
| <b>Location End:</b>        | Labor and Delivery  |
| <b>Transporter name:</b>    | Nurse Jackie        |
| <b>Transporter phone #:</b> | 800-555-1234        |
| <b>Time of Alarm:</b>       | 05/03/2022 17:43:09 |

At the bottom of the callout box, there are two buttons: "Accept" (green) and "Edit Bypass" (blue). Below the floorplan, there is a text input field containing "Labor and Delivery".

At the bottom of the interface, there is a copyright notice: "© 2022 - McRoberts Technologies | build version: 8122".

When the time allotted for the bypass is almost expired, a “BYPASS EXPIRING” alarm is triggered.

## LABOR AND DELIVERY

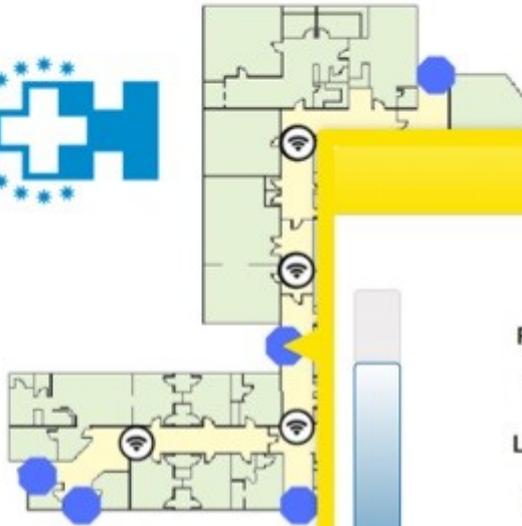

Labor and Delivery

### BYPASS EXPIRING

170 secs left

**Patient Name:** Davis, Brad

**Tag Number:** B2DE55

**Location Start:** Labor and Delivery

**Location End:** Labor and Delivery

**Transporter name:** Nurse Jackie

**Transporter phone #:** 800-555-1234

**Time of Alarm:** 05/03/2022 17:58:06

Accept

Edit Bypass

Click "Accept" to clear the alarm from the screen. The timer will continue to run out until the bypass expires or is complete.

Click "Edit Bypass" to edit the bypass. Changes can also be made on the patient card.

When the timer reaches zero, a "BYPASS EXPIRED" alarm is triggered. All bypass alarms (except BYPASS COMPLETE) allow you to edit the bypass. Changes can also be made on the patient card.

MyChild MyChild

Home Patients Tags Floorplans Reports Admin Help Logout

### LABOR AND DELIVERY

McRoberts HOSPITAL

**BYPASS EXPIRED**

|                             |                     |
|-----------------------------|---------------------|
| <b>Patient Name:</b>        | Davis, Brad         |
| <b>Tag Number:</b>          | B2DE55              |
| <b>Location Start:</b>      | Labor and Delivery  |
| <b>Location End:</b>        | Labor and Delivery  |
| <b>Transporter name:</b>    | Nurse Jackie        |
| <b>Transporter phone #:</b> | 800-555-1234        |
| <b>Time of Alarm:</b>       | 05/03/2022 17:55:36 |


Accept Edit Bypass

RN needs more time, edit bypass required

© 2022 - McRoberts Technologies | built

First, click the "Accept" button, then enter a message and click the green arrow to accept the alarm. The Bypass Expired alarm only happens if the time limit has run out and the patient has not passed through the second bypass designated door. If the patient arrives on time, you will receive a Bypass Complete alarm. This will show up in the upper right corner as a notification as it is a level one alarm. Click on the notification to view the alarm detail.

## LABOR AND DELIVERY

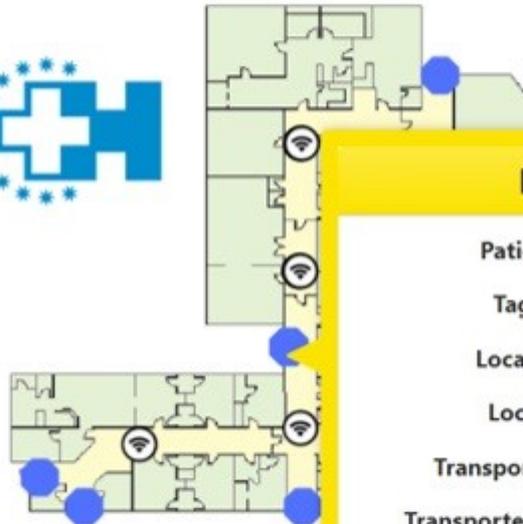

Labor and Delivery

## BYPASS COMPLETE

**Patient Name:** Davis, Brad  
**Tag Number:** B2DE55  
**Location Start:** Labor and Delivery  
**Location End:** Labor and Delivery  
**Transporter name:** Nurse Jackie  
**Transporter phone #:** 800-555-1234  
**Time of Alarm:** 05/03/2022 17:44:29

When you complete a bypass by clicking “Complete,” you are acknowledging that the patient has arrived at the destination unit. The patient status will return to "Active".

For additional help, call 1-800-776-7328 and press 2 for technical support.

[Click here to download PDF.](#)

---

Revision #15

Created 1 February 2022 16:55:04 by Admin

Updated 12 July 2022 16:20:52 by Anna Olt