

Alarms in MyChild6

Alarms in MyChild6

Patient Alarms are alarms that are triggered by assigned tags under certain conditions. Patient alarms show on your floorplan and point to the device that triggered that alarm or saw the tag last. The alarm includes the patient's name, tag number, and last known location.

Level 1 alarms notify you about events that could lead to a security breach. Level 1 alarms are indicated by pop-up notifications that disappear after 30 seconds unless you click "View Alarm." The alarm is displayed in the floorplan view. The floorplan view has priority over all other pages. Make sure you complete whatever you are working on before clicking on "View Alarm" because your work will not be saved.

Level 2 alarms also notify you about events that could lead to a security breach but are more serious than the events that trigger a level 1 alarm. It is recommended that security protocols be followed. There are no pop-up notifications. The floorplan view with the alarm automatically takes over your screen; therefore, whatever you are working on will not be saved. Audio is included.

Level 3 alarms notify you about events that could be security breaches. It is recommended that security protocols be followed. There are no pop-up notifications. The floorplan view with the alarm automatically takes over your screen; therefore, whatever you are working on will not be saved. Audio is included.

Revision #3

Created 11 February 2025 17:51:45 by Anna Olt

Updated 12 February 2025 16:38:38 by Anna Olt